



PerfectIM Systems


PerfectIM offers reliable performance and the ability to create accurate dental impressions without the need for inconvenient, uncomfortable retraction cord. Because these materials are hydrophobic, they will displace fluids, eliminating the need to air dry. Precise, easy to work with, and available in a variety of working/setting times, PerfectIM is ideal for bite registrations, dentures, crowns, bridges, veneers, inlays, onlays, and nightguards.

Perfect Impressions, No Cord!

The H&H (Hydraulic & Hydrophobic) cordless technique can easily save you an average of 15 minutes per “crown & bridge” appointment, while producing exceptionally accurate results. It is a two step system which eliminates the time consuming task of packing retraction cord, and therefore any possible post traumatic discomfort.



H&H Cordless Technique

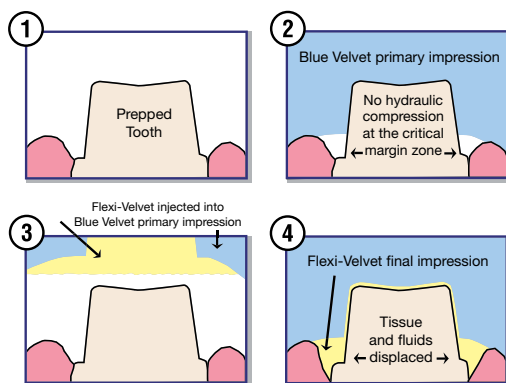

To order a demonstration CD or DVD of this revolutionary technique call, 888-JMORITA (566-7482). To view the demonstration online, visit our website at jmoritausa.com, click on products, then product demonstrations.

Features and Benefits

- Vinyl Polysiloxane - displaces fluids
- Thixotropic (Except SnoWhite & Final Wash)
- Radiopaque
- Cartridge dispensed
- Immersible in aqueous disinfectants
- Taste and odor free
- “Highly recommended” by The Dental Advisor in an evaluation study

To order, or for more information, call 888-JMORITA (566-7482) or contact your dealer.

PerfectIM Systems Product Selection Guide

PRODUCT	90 Second Blue Velvet®	30 Second Blue Velvet®	90 Second Flexi-Velvet®	30 Second Flexi-Velvet®	90 Second SnoWhite™	30 Second SnoWhite™	Single Phase Body™	Final Wash™
Working Time	90 Sec.	30 Sec.	90 Sec.	30 Sec.	90 Sec.	30 Sec.	2.5 Min.	2.5 Min.
Setting Time	2 Min.	60 Sec.	2 Min.	60 Sec.	2 Min.	60 Sec.	2.5 Min.	2.5 Min.
Durometer	80	80	50	50	50	50	60	50
PROCEDURE	Match your procedure and preferred working and setting times with the appropriate impression material.							
• Crown & Bridge • Full or Partial Dentures with FAST WORK/SET	1			2		2		
• Crown & Bridge • Full or Partial Dentures with SLOW WORK/SET	1		2		2			2
• Bite Registration with FAST WORK/SET		3						
• Bite Registration with SLOW WORK/SET	3							
• Crown & Bridge with SINGLE PHASE							3	
• Denture Reline			2		2			
• Customize a Stock Tray	3							

Chart legend: 1 = first step, 2 = second step, 3 = single step

No Doubt About A Bite®

The PerfectIM Systems 30 Second Blue Velvet has a whipped cream texture, which eliminates running, dripping, and gagging. It is the ideal material for bite registrations, producing fast, extremely accurate interocclusal records. The high durometer (80) of 90 Second Blue Velvet produces a hard, plaster-like set and eliminates the need for retraction cord when used with Flexi-Velvet or SnoWhite in the H&H cordless technique. As a second step companion to Blue Velvet, Flexi-Velvet can be used to capture perfect margins. SnoWhite can also be used as a second step in this technique; it is an extremely fluid material and is recommended for normal sulcus depth crown and bridge impressions, as well as for inlay and onlay impressions. 90 Second SnoWhite is also recommended for denture relines and as a final wash for denture impressions. Peel-Away Die Spacer is designed for use on dies produced from H&H impressions; it is easily removed and a final check for fit can be made on the original clean die.

Replacements

Single Phase Body can be used as a replacement for Impregum™. Final Wash can be used as a replacement for Permadyne™.

Ordering Information

Cartridge Kits

Each kit contains (3) 50ml cartridges and 12 mixing tips

90 Second Blue Velvet®	21-300-550
30 Second Blue Velvet®	21-300-500
90 Second Flexi-Velvet®	21-300-650
30 Second Flexi-Velvet®	21-300-625
90 Second SnoWhite™	21-300-950
30 Second SnoWhite™	21-300-925
Single Phase Body™	21-300-700
Final Wash™	21-300-800

Related Products and Accessories

PerfectIM Putty Type (C & B / 400g each)	21-300-050
PerfectIM Tissue Mimic (2 cart. /100ml)	21-301-100
Cartridge Dispensing Gun	21-300-015
Mixing Tips Large (12)/(50)	21-300-044/21-300-044L
Mixing Tips Small (12)/(50)	21-300-047/21-300-047L
Intraoral Tips (100/pkg.) for Flexi-Velvet, SnoWhite, and Final Wash	21-300-045
Peel-Away Die Spacer Complete Kit	21-400-000

Impregum and Permadyne are registered trademarks of 3M ESPE. PerfectIM, Blue Velvet, Flexi-Velvet and No Doubt About A Bite are registered trademarks of J. Morita USA, Inc.

For some procedures, you may have several choices of materials. Working times are for product stored at ambient temperatures of 73° F / 23° C. Higher temperatures will shorten, while lower temperatures will lengthen working time.

The specified intraoral set times are the minimum time the material should remain in the mouth based on an oral temperature of 93° F / 34° C.



Blue Velvet Bite Registration



Flexi-Velvet Impression



SnoWhite Impression

Clinical photography courtesy of Dr. Jeffrey C. Hoos, Stratford, CT

J. MORITA
USA INC.

WWW.JMORITAUSA.COM