Instructions for Use

Please read these instructions carefully before using FOUNDATION.

Product Overview

This product is made of atelo-collagen to minimize antigenicity. The atelo-collagen has been cross-linked by heat treatment in order to achieve biocompatibility. The product is processed in a sponge block and then it is formed into a bullet shape for easy placement in the extraction socket. It consists of fibrillar and heat-denatured atelo-collagen. Fibrillar atelo-collagen provides scaffolding for surrounding cells and the heat-denatured atelo-collagen stimulates infiltration of the cells in the product. Contents are sterile and non-pyrogenic. Clinical experience suggests the use of this device may lead to reduced osseous resorption.

Indication for Use

FOUNDATION is a collagen-based bone filling augmentation material for use in the filling of extraction sockets.

Contraindication

This product uses proteins of bovine origin as raw material and should not be used in those patients who have a known history of hypersensitivity.

Warnings

1. Use of this product should be discontinued immediately if the patient displays any symptoms such as rash, hives, or other allergic reactions.
2. Do not touch the material with your bare hands.

Precautions

1. Do not resterilize or reuse. This product is sterile and intended for single use only.
2. Do not use if the package and/or product is damaged.
3. Once the unit package has been opened, discard if not used immediately.
4. Use of this product in patients predisposed to asthma, hives, or other allergic reactions should be evaluated prior to use.
5. Use of this product in patients who have just given birth or who are nursing a baby should be evaluated prior to use.
6. The safety of this product in pregnant patients has not been studied or established, therefore, use in pregnant patients should be evaluated for associated risks.
7. This product has no antimicrobial properties. Any decision regarding the use of an antibiotic or other therapies when using this product should be made based upon the patient's risk of infection.
8. Applying excessive amounts of the material may cause tenderness and irritation of the socket.
9. Secure FOUNDATION by suturing, or by some other means. Failure to do so can cause dislodgement of the material which can present a choking hazard.

Caution: US Federal Law restricts this device to sale by or on the order of a dentist.

How to Open the Container

1. Open aluminum package at TEAR HERE and remove plastic container.
2. While holding the edge of the container, remove FOUNDATION using a sterile instrument. Do not touch FOUNDATION with your bare hands.

Instructions for Use

1. If the extraction socket is bleeding excessively, achieve hemostasis with gauze or other appropriate material.
2. Where appropriate, curette the gingiva in the socket to stimulate the infiltration of the surrounding blood into the product.
 - a) If the product is to be used in an oroantral fistula that has been left untreated for an extended time resulting in the mucous epithelium covering the fistula, strip the mucous epithelium from the fistula before insertion of FOUNDATION.
 - b) If the product is to be used in a socket with chronic inflammation, post-fracture or other disease, remove the edge of gingiva from the inside of the socket to the top of the alveolar bone and curette the unhealthy granulation tissue before application of FOUNDATION. Failure to do so may dislodge FOUNDATION from the socket and the alveolar ridge may cave in.
3. Select a slightly larger size of FOUNDATION than the socket to be filled. Remove FOUNDATION from the container and place it in the socket with a sterile instrument.
4. Secure FOUNDATION by suturing, or some other means, to prevent its dislodgement from the socket.
5. As necessary, immediate rinsing, gargling, or other appropriate treatment should be considered to minimize the risk of infection.

Storage

Keep dry. Keep away from heat. Avoid direct sunlight. Do not store under conditions of extreme humidity or room temperatures exceeding 86° F (30° C).

Packaging

Item/Product Number	Dimensions (dia. x length)	No. of packages (per box)
S (Small) 27-500-100	Ø 8 x 25 mm	10
M (Medium) 27-500-200	Ø 15 x 25 mm	5
Assortment Pack 27-500-150	(includes both)	3 Small, 3 Medium

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