



## User's Instructions for CP5-O <with Fiber Optic> Coupling

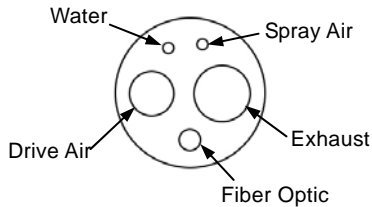

\*Before use check the following items:

Does the connection end of the tube match the coupling? This coupling cannot be connected to a tube that does not have light capability. This coupling cannot be connected to a 4-hole tube.

### 1. Connect Coupling

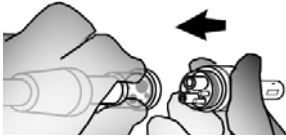

Line up the projections on the coupling with the indentations in the tube and then tighten up the cover nut.

#### ⚠ NOTE

- *Before connection, make sure the tube connector is clean and free of debris.*
- *The light will lose its brightness if the end of the fiber optic cable is scratched or otherwise damaged by being dropped for instance.*



Tighten securely with the wrench provided.

#### ⚠ NOTE

- *Air or water may leak if the coupling is not tightened up enough.*

\*Before first using the coupling or if the handpiece-coupling connection gets stiff, take the nozzle off the AR Spray can and lightly spray the coupling's o-rings.

### 2. Cleaning

Wipe the outside cover with ethanol.

#### ⚠ NOTE

- *Coupling must NOT be autoclaved or cleaned ultrasonically.*
- *The coupling could be damaged if it is soaked in ethanol or cleaned with a strong, corrosive solution.*
- *Clean both ends of the light guide if the light seems to have lost its brightness.*

### 3. Specifications

Model : CP5-O <with Fiber Optic>  
ISO 9168 Type B Coupling

Manufactured by **J.MORITA MFG. CORP.**

PUB. H5062-EA Printed in Japan