



Doctor Name: _____
Office Address: _____
City, State, Zip: _____
Phone: _____
Email: _____
J. Morita Sales Representative Name: _____
Or
Dealer Representative Name, Company & Phone Number: _____

Circle the letter for the correct answer on each question.

1. What does “H and H” stand for?
 - a. Hydraulic and Hydrophilic
 - b. Hydraulic and Hydroponic
 - c. Hydraulic and Hydrophobic

2. “Blue Velvet” can be used for:
 - a. Primary Impressions
 - b. Bite registration
 - c. Both a and b

3. What is “SnoWhite” best suited for?
 - a. Inlays and onlays
 - b. Supragingival impressions
 - c. Both a and b

4. The purpose of the “Free Access” cheek retractors is:
 - a. Ensure maximum intercuspation of teeth
 - b. Improve visibility
 - c. Both a and b

5. During the primary impression, it is very important to physically ensure maximum intercuspation by upward placing pressure on the mandible.
 - a. True
 - b. False

6. It is critical to capture the margins of the prepared tooth with Blue velvet.
 - a. True
 - b. False

7. It is not recommended to take an impression without the use of a tray.
 - a. True
 - b. False

8. A good method of judging when the Blue Velvet is set is to:
 - a. Ask the patient
 - b. Check the material on the tip of dispensing gun
 - c. Wait until the material changes color

9. It is okay to see some blue showing through when you have taken the final impression with Flexi-Velvet or SnoWhite.
 - a. True
 - b. False

10. The following methods can be used to evaluate maximum intercuspation:
 - a. Hold up to the light and look at areas of the unprepared teeth
 - b. Visually examine opposite arch during impression with cheek retractors in place
 - c. Both a and b

11. What are the two available choices for working time for Blue Velvet, Flexi-Velvet, and SnoWhite?
 - a. 30 and 90 seconds
 - b. 60 and 120 seconds
 - c. 120 and 240 seconds

12. You need to use a very liberal amount of Flexi-Velvet or SnoWhite in order to obtain an accurate final impression.
 - a. True
 - b. False

13. A supragingival impression can be obtained with Flex-Velvet or SnoWhite without the use of a retraction chord, but a chord is necessary for subgingival impressions.
 - a. True
 - b. False

14. PerfectIM Systems impression material is:

- a. Vinyl Polysiloxane
- b. Rubber based
- c. Silicone

15. The Blue Velvet material should initially be placed on the arch opposite the prepared tooth.

- a. True
- b. False

Please fax completed tests to: (949) 581-8790.

For passing tests, CE certificates will be mailed within 3 weeks to the address provided on this form. Questions? 888-JMORITA (566-7482).